

# User manual

## MANO EDU

NE



Please read this user manual carefully prior to installation, use and maintenance for your own safety and in order to avoid damages on your system.



# Content

|                                                       |           |
|-------------------------------------------------------|-----------|
| <b>I. General Information .....</b>                   | <b>5</b>  |
| I.I General Information – valid for MANOEDU1 .....    | 7         |
| I.II General Information – valid for MANOEDU1LV ..... | 9         |
| <b>II. Safety instructions .....</b>                  | <b>10</b> |
| <b>III. Instructions .....</b>                        | <b>11</b> |
| Scope of supply .....                                 | 11        |
| Accessories ( optional ) .....                        | 11        |
| Commissioning .....                                   | 11        |
| Charging the battery .....                            | 11        |
| Power indicator .....                                 | 11        |
| Setting up MANO EDU .....                             | 12        |
| Distance camera setting up .....                      | 13        |
| Attaching the anti-glare shield .....                 | 13        |
| Description of the controls .....                     | 14        |
| Operation .....                                       | 15        |
| Turning ON/OFF .....                                  | 15        |
| Home-/menu button .....                               | 15        |
| Zoom .....                                            | 15        |
| Move the magnified image .....                        | 16        |
| Full-page reading (A4) .....                          | 16        |
| Color mode .....                                      | 16        |
| Orientation aids in the magnified image .....         | 17        |
| Image position .....                                  | 17        |
| Line ruler and line mask .....                        | 17        |
| Overview mode .....                                   | 17        |
| Live Panning .....                                    | 17        |
| The menu bar in zoom mode .....                       | 18        |
| Lighting .....                                        | 18        |
| Micrograph function .....                             | 18        |
| Brightness .....                                      | 18        |
| Adjusting the contrast (in semi color mode) .....     | 18        |
| Volume .....                                          | 18        |
| Freeze frame function .....                           | 19        |
| Save freeze image .....                               | 19        |
| Save multiple pages as a PDF .....                    | 19        |
| Autofocus .....                                       | 20        |
| Toggle image view .....                               | 20        |
| MANO EDU without a second display .....               | 20        |
| Switching the viewing mode .....                      | 20        |
| Split screen control .....                            | 20        |
| Toggle between distance view and mirror view .....    | 20        |
| Document view of the distance camera .....            | 21        |
| MANO EDU with a second display .....                  | 21        |
| Switching the viewing mode .....                      | 21        |
| Using the camera while viewing documents .....        | 22        |
| Micrograph function .....                             | 22        |

|                                                          |           |
|----------------------------------------------------------|-----------|
| Read aloud function (text to speech) .....               | 22        |
| Reading a full page (up to A4).....                      | 22        |
| TTS of the magnified image .....                         | 22        |
| Operation in reading mode .....                          | 23        |
| Picture mode .....                                       | 23        |
| Segmentation activated .....                             | 23        |
| Segmentation deactivated .....                           | 24        |
| Text mode .....                                          | 24        |
| Segmentation was activated in image mode.....            | 24        |
| Segmentation was disabled in image mode.....             | 24        |
| The main menu .....                                      | 25        |
| Open and copy document / picture .....                   | 25        |
| Manage files on the device.....                          | 25        |
| Copy files from PC/laptop or SD card to the device ..... | 26        |
| Copy saved files to an SD card.....                      | 26        |
| Delete all files from the device .....                   | 26        |
| Deleting files from a folder.....                        | 26        |
| Multiple selection .....                                 | 26        |
| Open pictures .....                                      | 27        |
| Open documents .....                                     | 27        |
| View and read aloud text files .....                     | 27        |
| View and read aloud PDF files .....                      | 28        |
| Settings.....                                            | 28        |
| Options page .....                                       | 28        |
| Device information.....                                  | 29        |
| HDMI output .....                                        | 29        |
| Hardware reset.....                                      | 30        |
| Handling of the battery .....                            | 30        |
| Changing the battery .....                               | 30        |
| <b>IV. Troubleshooting .....</b>                         | <b>31</b> |
| <b>V. Warranty information .....</b>                     | <b>32</b> |
| <b>VI. Technical details .....</b>                       | <b>33</b> |

## **I. General Information**

**Important notes on the operation of MANO EDU!**

**Please read them carefully prior to the operation of the device!**

### **Notes on the electromagnetic compatibility (EMC)**

MANO EDU is suited for the use in all set-ups including the domicile that are connected directly to a public power supply for buildings with residential purposes.

MANO EDU is designed for the use in the indicated electromagnetic environment. The user should ensure the use in the indicated environment.

MANO EDU operates on HF-Energy exclusively for its own function. Therefore the emitted interference of high frequency disturbances is very low and the interference with other appliances operated close-by is improbable. When operating the system simultaneously with other systems, however, the possibility of disturbances of the system or other systems should not be neglected.

For this reason it should be ensured that the system is not placed directly beside or on top of another system. If the set-up of this system in the proximity of medical, measuring devices cannot be avoided, the operator of these devices has to be informed that it is necessary to observe the performance of the system to control the conventional use of the system in the chosen set-up.

Electronic devices are sensitive towards electric discharge and towards high frequency electromagnetic fields.

### **Attention:**

In order to avoid high frequency disturbances (e.g. microwaves) the operator should respect a minimum distance of one meter/three feet to portable and mobile HF-telecommunication devices.

Please also see chapter „Safety advice“ on page 7 or on page 9.

### **Storage of the user manual**

The user manual is an essential part of the system. Please keep the manual in a safe place for future reference.

### **Product description**

MANO EDU is an electronic magnifier for enlarging text and images and for converting and reproducing printed text into synthetic speech.

The FullHD 12.5" (approx. 31.8 cm) touch display on a tiltable stand offers an extremely sharp and high-contrast display.

- Approx. 1x to 60x, continuous magnification thanks to the innovative and state of the art camera technology
- Real-time magnification and text recognition of an entire A4 page
- Split screen (2 camera images at the same time)
- Reading camera with integrated LED lighting and distance camera
- easy to fold with a practical handle

## **Cleaning of the system**

- Before cleaning, disconnect the unit from the power supply.
- Clean the device with a mild household cleaner and a microfiber cloth.
- Do not allow any liquids to enter the device.

## **Special advice for transport and storage**

For safety reasons, the unit must only be shipped in the original packing. The unit must be stored in dry and closed rooms in the original packing.

The following conditions for transport and storage have to be respected:

- Air pressure 700 to 1000 hPa
- Temperature range + 5 °C to + 50 °C
- Relative humidity 30% to 80%

## **Important note on reading, capturing and playback of medication leaflets**

Dosage and quantity information from medication leaflets, read from the screen or, if a read-aloud function in your system is available, played back, must always be checked and re-confirmed independently! Always ask your doctor, pharmacist or a person you trust about dosing of medications!

## **Power supply**

Only use the original power supply which is included in the delivery by Reinecker Vision GmbH.

**Caution: The use of any other power supply besides the original, might lead to an increased electro-magnetic emission and to product damage.**

## **Disposal or transfer of the device**

This equipment may retain user settings and stored images if disposed of or transferred. As described on page 28, go to the device information in the settings and then to “Reset to factory settings” to remove all user settings. To remove all personal content from the device, follow the procedure in “Deleting all files from the device” on page 31.

## **I.I General Information – valid for MANOEDU1**

The order code of your device can be found on the type label on the device or on the packaging.

### **Purpose**

This electronic magnifier with voice output is a medical device to assist with the reading and magnifying of printed text and pictures. Visually impaired people can compensate for their loss of visual acuity by using this device.

### **Medical device**

This device is classified as a medical device, class I according to (EU) 2017/745, Annexe VIII, standard 13. For the distribution to countries outside the EU the respective country-specific guidelines have to be regarded.

### **Declaration of conformity, CE mark**



You can request a copy of the declaration of conformity in written form from Reinecker Vision GmbH, Sandwiesenstr. 19, 64665 Alsbach-Hähnlein or view it at [www.reineckervision.de/eu-doc](http://www.reineckervision.de/eu-doc)



### **Evolution and lifetime of products**

The Medical Device Directive regulates by law that manufacturers of medical devices shall determine a period of safety operation of a product. We fulfill hereby legal requirements and determine - for formal reasons - a period of three years after date of first purchase.

**Note:** The mentioned life span does not reflect the complete life span of the system. It is rather an indication on when the product might be obsolete and substitution might be recommended due to new product developments.

### **Combination with other medical devices**

If the operator/user wants to operate this device in conjunction with other life sustaining medical devices, please get in touch with Reinecker Vision GmbH or with your local dealer, prior to operation.

### **Safety advice**

The device has to be set up and operated in closed and dry rooms only. Do not plug the device into an electrical circuit that is also being used for any life sustaining or support systems. The device may not be used in the patients environment.

This device should not be operated in explosion-endangered environments. This device should only be installed in conjunction with a state-of-the-art electrical circuit.

### **Precautions by alteration of the performance of the device**

Should any of the technical features or performances change (failure, malfunction etc.), the device must be turned OFF and the battery must be taken out. **Please do not switch ON the device again.** Refer directly to your authorized dealer or to Reinecker Vision GmbH Service Center.

## Reinstatement, installation and maintenance

This device may be technically repaired and reconditioned by Reinecker Vision GmbH only or by an authorized technician. The device may only be used again after a safety check, thorough cleaning and disinfection.

## Environmentally friendly disposal (Directive 2012/19/EU)



Within the EU the respective national implementations of the Directive 2012/19/EU for the sustainable re-use and proper recycling of electrical and electronic equipment applies.



For countries outside the EU (e.g. Switzerland), the respective national implementations for the regulatory requirements for environmentally friendly disposal and recycling applies.

## List of symbols used

|  |                              |  |                          |
|--|------------------------------|--|--------------------------|
|  | Manufacturer                 |  | Country of manufacture   |
|  | Model number                 |  | Catalogue Number         |
|  | Serial number                |  | Caution                  |
|  | Medical device               |  | Unique Device Identifier |
|  | Consult instructions for use |  | Temperature limit        |

## **I.II General Information – valid for MANOEDU1LV**

The order code of your device can be found on the type label on the device or on the packaging.


### **Purpose**

MANO EDU is a system for magnifying text and pictures with voice output.

### **Declaration of Conformity (DoC), CE/FCC mark**



You can request a copy of the declaration of conformity in written form from Reinecker Vision GmbH, Sandwiesenstr. 19, 64665 Alsbach-Hähnlein or view it at [www.reineckervision.de/int-doc](http://www.reineckervision.de/int-doc)

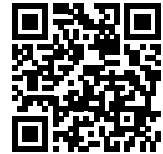

### **Safety advice**

The device may only be set up and operated in closed and dry rooms. The device should not be operated in explosion-endangered environments. The device may only be connected to a state-of-the-art circuit.

### **Repair and maintenance**

The device may only be serviced and repaired by Reinecker Vision GmbH or an authorized dealer.

### **Environmentally friendly disposal**



The device contains electronic components.  
To avoid environmental damage, the respective country-specific laws and regulations for the disposal of the device must be respected.

## **II. Safety instructions**

Please respect the following safety instructions:

1. Do not plug the AC adapter into a faulty electrical outlet. Doing so may cause electrical shock or fire.
2. Do not touch the AC adapter with wet hands.
3. Only use the original power supply which is included in the delivery by Reinecker Vision GmbH. The use of any other equipment might lead to a product damage.
4. If the system is equipped with batteries, only original parts from Reinecker Vision GmbH should be used for replacement.
5. Do not expose the system to direct sunlight or extreme heat or cold temperatures, as this would harm the picture quality.
6. Do not operate this device below +10°C or above +35°C.
7. Keep liquids away from the system. Only operate the device in a dry environment.
8. The device should not be operated next to medical devices (patients environment).
9. Do not press or drag objects across the touch display. Do not lean on the screen or lay heavy objects on the device. This may cause damage to the device.
10. Do not try to repair or to dismantle the system on your own. That would immediately void the warranty. If you need any technical assistance please contact your distributor or Reinecker Vision GmbH.
11. Always turn the system OFF and disconnect it from the power supply before cleaning it. Only use a soft and slightly damped cloth. Only use a mild household detergent. Do not use alcohol containing or strong detergents.

### III. Instructions

#### Scope of supply

- MANO EDU
- carrying bag
- anti-glare shield
- Cleaning cloth
- Power supply with adapters
- HDMI cable
- quick reference guide
- user manual

#### Accessories ( optional )


- MultiView screen
- spare battery
- battery charging station

#### Commissioning

Before using the device for the first time, you must fully charge the battery.

#### Charging the battery

Connect the supplied power cable to the charging socket on the device, then connect the power adapter to a mains socket to start the charging process. A complete charging process takes approx. 3 hours and enables an operating time of up to approx. 5.5 hours (depending on the operating mode).

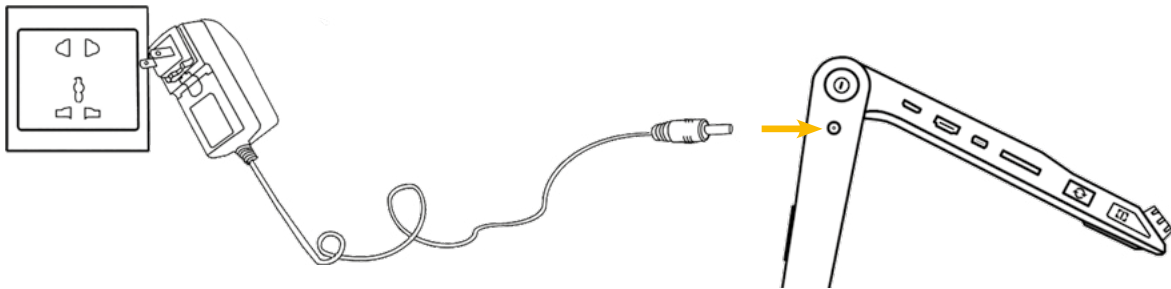

For more information, see „Handling of the battery“ on page 30 .

**Caution:** The AC adapter supplied with the unit is designed to operate this system only. Do not use any other AC adapter.

#### Power indicator

The symbol of the power button lights up in different colors, which can be used to indicate different charging states. The following status indicators are available:

Device switched OFF, power supply connected:

- Green power indicator: device is charging
- Power indicator OFF: device is fully charged

Device switched ON:

- Green power indicator: sufficient battery charge or power supply connected
- Red power indicator: battery almost empty, device needs to be charged
- Yellow power indicator: device is in standby

If the device is switched ON, you can see the battery level on the display by briefly pressing the power button.

## Setting up MANO EDU

Place the device on a flat surface, press the tray to make the device stable and pull up the handle at the same time.

Keep pressing the tray and unfold the arms to a suitable angle.



Unfold the display panel to a comfortable reading height.



## Distance camera setting up

The distance camera is located on the right side of the stand and can be rotated into different positions. Rotate the camera arm counterclockwise backwards until it is in the desired position. The yellow ring can be used to rotate the distance camera to different positions, so that it can also be used as a mirror, for example. To learn more about the different application options, see „Toggle image view“ on page 20.



## Attaching the anti-glare shield


Depending on the ambient light, distracting reflections on your reading material can make reading difficult. In this case, you can attach the anti-glare shield to the back of the stand. Peel the sticker cover off the back of the shield and stick it to the back of the device as shown in the illustration.



**Note:** The anti-glare shield can be easily removed from the device and reattached if necessary.

Please keep the sticker cover and use it to cover the sticker when you remove the anti-glare shield. If the adhesive strength decreases due to dust or other dirt, you can carefully clean the adhesive points with water.


## Description of the controls

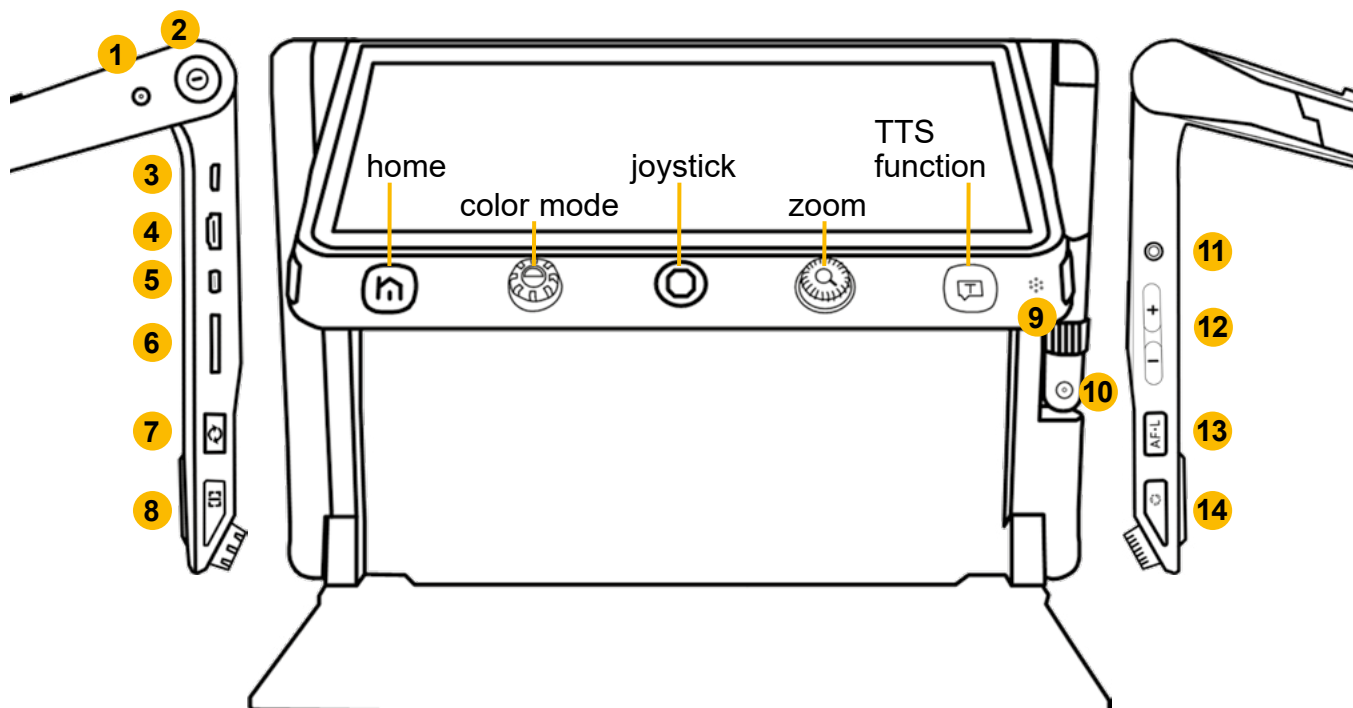

- 1) DC port
- 2) ON/OFF
- 3) HDMI input
- 4) HDMI output
- 5) USB port

- 6) SD card slot
- 7) camera switch
- 8) Line/Mask button
- 9) microphone
- 10) distance camera

- 11) headphone / loud-speaker socket
- 12) volume +/-
- 13) autofocus
- 14) freeze frame

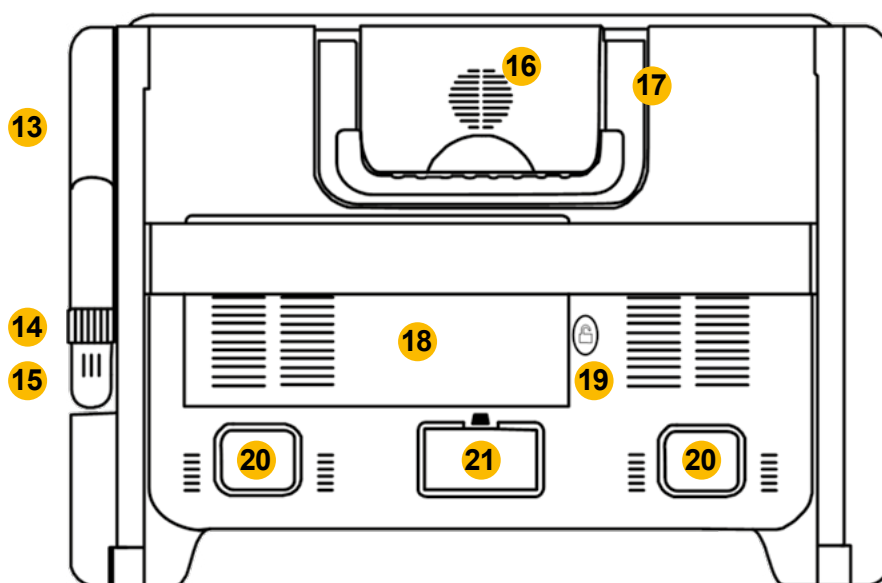

- 13) camera arm
- 14) rotatable camera ring
- 15) distance camera

- 16) speakers
- 17) handle
- 18) replaceable battery

- 19) battery lock
- 20) LED lighting
- 21) main camera

## Operation



MANO EDU can be operated by the use of buttons and by the use of the touch screen monitor. Menu items on the screen can be selected either by tapping on the item on the touch screen or by using the joystick. In case you have selected the symbol with the joystick, press the TTS button to confirm your selection.

## Turning ON/OFF



Press and hold the ON/OFF button for 3 seconds to turn on MANO EDU. When the unit is ON, press the ON/OFF button to check the current status of the battery capacity.

When the unit is ON, press and hold the ON/OFF button to turn OFF MANO EDU. A menu appears in which you can choose to either switch MANO EDU to standby mode or to turn it OFF. No operation for 3 seconds will automatically switch to the highlighted mode.

## Home-/menu button



If you press the home button for approx. 2 seconds, you will get to the main menu in which you can select the following functions:

- Magnifier
- Text to speech (TTS)
- Open document / picture
- Settings

The selected item is highlighted and you can tap on the desired icon on the touch-screen (or use the joystick to select and the TTS button to confirm).

Briefly press the home button to display the menu bar at the bottom of the screen when you are in magnification or reading mode. Depending on the current mode, the menu bar contains different items such as Lighting ON/OFF or the reading speed. You can also open the menu bar by sliding your finger up over the display from the bottom of the screen.

If you do not perform any operation for 5 seconds or briefly press the home button, the menu bar will be closed.

**Note:** In TTS mode, long press the home button to stop reading.

In menus you can go back or close the menu with the home button.

## Zoom



Turn the zoom button clockwise to zoom in and counterclockwise to zoom out.



Alternatively, you can adjust the magnification with two fingers on the touch-screen by moving your fingers apart (zoom in) or towards each other (zoom out).

## Move the magnified image

At higher magnification levels you can navigate through the image by sliding your finger over the touchscreen or moving the joystick. This enables you to read comfortably without having to move the reading material under the device.



### Full-page reading (A4)

The device can capture an entire A4 page in order to zoom in and move the image on the screen without having to move the reading material.

Press the home button to display the quick control bar. Tap the A4 icon to activate full page mode.



Alternatively, you can press and hold the camera switch button on the left side to switch to full page mode.

Tap the snapshot icon or press the freeze button to capture the full page image.



To rotate the picture, press the freeze button or tap on the rotate symbol in the menu bar.

Press the home button for 2 seconds or tap the exit icon to close full page reading.



Press the TTS button to access the reading aloud function. You can find out more about operating MANO EDU in text-to-speech mode under „Operation in reading mode“ on page 23 .

## Color mode



Full color mode is suitable for displaying pictures or maps. In semi color mode (a combination of foreground and background color), texts can be read with high contrast.

Rotate the color mode button to cycle through all activated semi color combinations. Press the color mode button to quickly switch between true color mode and the last selected semi color.

You can access the semi color settings by holding down the color mode button for approx. 5 seconds. There you can activate/deactivate the semi color combinations by tapping (or use the joystick to select and the TTS button to confirm).

The following semi color combinations are available:



White and black



Yellow and black



White and blue



Blue and yellow



Green and black



Black and magenta



Black and blue



White on Red



Red on black

Tap the exit icon to exit the custom colors page (or press the home button).

**Note:** The custom colors page can also be accessed from the settings menu.

## Orientation aids in the magnified image

### Image position

Double tap the center of the image to re-center the image (or push the joystick in any direction and press the zoom in button at the same time).

Double-tap the at edge of the display to jump to the end of the original image (or press the joystick twice quickly in the desired direction).

### Line ruler and line mask



Press the orientation aids button on the left to toggle between the different modes for line ruler and line mask.

You can change the color of the line ruler by holding down the orientation aids button and rotating the color mode button.

Hold down the orientation aids button and move the joystick to change the size and position of the line ruler and line mask.

### Overview mode



When you are at a higher zoom level, briefly press the magnification button to see the original-sized image. A yellow box marks the section of the magnified image. Use the joystick or a finger on the display to move the position of the box.

If you want to return to the previous position, turn the magnification button in any direction. To center the magnified image section, move and hold the joystick in any direction and press the zoom button.

Briefly press the magnification button again (or double-tap the box) to return to the magnified image.

### Live Panning



In a higher zoom level, use the slide gesture (or use the Joystick ) to navigate the image for seeing more without moving the paper.

Push the joystick in any direction and turn the zoom button to display the bar for the panning speed.

To adjust the panning speed, turn the magnification button or adjust the slider on the screen with your finger to the desired position.

Do not press any button for 5 seconds or briefly press the home or TTS button or tap the inactive area to close the panning speed setting.

## The menu bar in zoom mode

Briefly press the home button in zoom mode or slide your finger upwards from the bottom of the screen to open the menu bar.

If you do not perform any operation for 5 seconds or briefly press the home button, the menu bar will be closed.

Select the desired icon on the touchscreen (or use the joystick to select and the TTS button to confirm).

## Lighting



Show the menu bar and tap the light bulb on the left to switch the entire lighting ON and OFF. By tapping on the lamps on the right, you can switch the left and right LEDs ON and OFF individually.


This can be helpful to minimize glare on the reading material.

## Micrograph function



This function is only available if you have a second display connected. For more information, see „Micrograph function“ on page 22 .

## Brightness

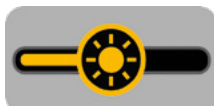

Open the menu bar and select the brightness symbol with your finger (or use the joystick). Slide your finger to the left or right (or turn the magnification button) to increase/decrease the brightness.

No operation for 5 seconds, briefly pressing the home button or tapping the inactive area closes the controller.

## Adjusting the contrast (in semi color mode)



The contrast is automatically optimized in the semi color mode, but can also be set manually. Press the color mode button while turning the magnification button to adjust the contrast.

Slide your finger (or rotate the magnification button) to increase/decrease the contrast.

No operation for 5 seconds, briefly pressing the home button or tapping the inactive area closes the controller.

## Volume



The button/speech guide volume and the TTS volume can be controlled separately.



To do this, use the volume buttons on the right side of the device or slide the slider on the screen with your finger to the desired position.

## Freeze frame function



Press the Freeze button on the right side of the device to create a freeze frame. Press the button again to return to live mode.

When viewing freeze images, you can change the magnification and color mode as described on page 15.

## Save freeze image



In the magnification mode, press and hold the Freeze button for 3 seconds to save the current image.



After saving, you have the option of recording a spoken label.

Tap and hold the microphone icon to start recording and release to stop (or press and hold the Read button to start recording and release, to end them).



You can listen to the recorded label by pressing the play icon (or by pressing the zoom button).

If you do nothing for 10 seconds, tap on the home button or on the inactive area to return to the freeze frame.

For the best possible recording quality, please speak loudly and clearly into the microphone to the right of the read button.

**Note:** The voice label function can be turned ON or OFF in the settings (see page 28 ).

You have about 2 GB of storage space available. If an error message appears while saving, the memory is full and you have to delete pictures in the gallery or transfer them to a microSD card.

Further information on the gallery can be found on page 25 .

## Save multiple pages as a PDF



This function is available in near vision and can be switched ON and OFF in the settings under "Document scan". If the function is activated, you will see a scan symbol on the right side of the screen when the menu is displayed. Tap the icon to enter scan mode (or press the freeze button).



Tap the arrow to record what is displayed on the screen. Turn the page and tap the arrow again and repeat the process until you have recorded all of the pages.



Then press the save icon or the home button. A pop-up appears where you can save the PDF or exit the menu without saving.

Long press the home button to exit the scanning process directly without saving a PDF.



In full page reading mode, you can rotate the image and crop it to the correct page size. Rotate the picture by tapping on the icon and change the size of the section by adjusting the size of the colored box with your fingers.

## Autofocus



In most cases, the autofocus works well and the image is sharp and easy to read. If this is not the case, you can refocus the image by holding your finger on the screen for 2 seconds (or pressing the autofocus button for 2 seconds).

Briefly press the autofocus button to activate or deactivate it.

If you want to write under MANO EDU, turn the autofocus OFF before you place your hand under the display to write something. This will prevent the hand or the pen from being in focus and not the writing.

## Toggle image view



MANO EDU offers various image views such as split screen or mirror view, depending on how the distance camera is being used and whether a second screen has been connected.

## MANO EDU without a second display

### Switching the viewing mode




Press the Shift key to toggle between the four different viewing modes.

The following viewing modes are available:




Near vision

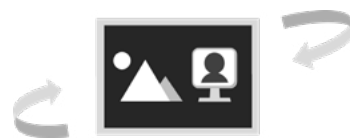

Distance view / mirror view



Horizontal split screen

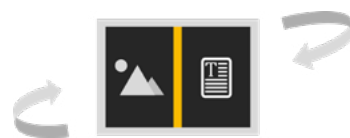

Vertical split screen

## Split screen control

In the split screen view you can see the image of both the near view and distance camera at the same time. A red frame highlights the active camera image in which the settings for magnification and color mode can be made.

If you would like to highlight the other camera image, tap on the camera image with your finger or briefly press the orientation aids button on the left.

You can adjust the proportions of the camera images by holding down the orientation aids button and moving the joystick up or down.

## Toggle between distance view and mirror view

Turn the yellow ring on the distance camera clockwise until the camera is pointing at you. From a certain position on, the view automatically changes from distance view to mirror view.



## Document view of the distance camera



The distance camera can also be used for reading and writing. To do this, turn the camera forwards until it is parallel to the surface and the camera lens is pointing downwards. In the distance view mode, open the menu bar and tap the mirror icon (or move the joystick to select and press the TTS button to confirm) to display the image correctly.



## MANO EDU with a second display

MANO EDU can be connected to a second display via the HDMI connection. The optionally available display can be plugged into the rear stand bar. This allows you to use both displays in a comfortable viewing position.



## Switching the viewing mode




Press the Shift key to toggle between the four different viewing modes.

The following viewing modes are available:



Near vision

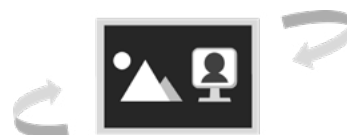

Distance view / mirror view



Near view on the main display



Distance view on the main display

If both camera images are displayed at the same time, a red frame highlights the active camera image in which the settings for magnification and color mode can be made.

If you would like to highlight the other camera image, tap twice quickly with your finger in a corner of the main screen (or push the camera switch button and move the joystick). Alternatively, you can change the active screen by using the menu bar.

## Using the camera while viewing documents



When viewing a document or picture (see page 25 ff.), or using the TTS function, the camera is still active in the second display. With the menu bar hidden, press the Shift key to activate the camera (a red frame marks the camera image). Press the Shift key again to switch to the active document.

**Note:** Advanced functions such as TTS mode, freeze frame, ruler and mask are deactivated in this viewing mode.

## Micrograph function



This function is only available if the images on the main monitor and the external monitor are identical.

Touch the magnifier symbol in the menu bar or press the magnification button for approx. 2 seconds to activate this function. You will now see the magnified image on the second display and the original image on the main screen with a yellow frame around the magnified area.

You can still magnify the image, change the color mode and move the image section with your fingers or the joystick.

Touch the magnifier symbol in the menu bar or press the magnification button for approx. 2 seconds to deactivate this function.

**Note:** Advanced functions such as TTS mode, freeze frame, ruler and mask are deactivated in this viewing mode.

## Read aloud function (text to speech)

There are two text recognition or OCR (Optical Character Recognition) modes that you can choose from before capturing. You can either capture an entire A4 page and have it read aloud or you capture just the text of the magnified image section.

## Reading a full page (up to A4)



Press and hold the TTS button for approx. 2 seconds and you will enter the text capture mode for a whole page. Here you can switch the lighting ON and OFF before capturing the page. To start reading, tap the text-to-speech icon in the bottom right corner or press the TTS button.

Press and hold the home button to exit the TTS mode.

## TTS of the magnified image



Press the TTS button to capture the displayed image section and switch to reading mode.

MANO EDU usually detects and adjusts the image alignment correctly. You can also manually align the displayed image if necessary. Press and hold the camera switch button until the menu bar appears. Here you can manually correct the image alignment. Then press the TTS button to reactivate the OCR function.

Press and hold the home button to exit the TTS mode.

## Operation in reading mode

A loading icon appears while the text is being captured. The capture time depends on the structure and amount of the text. When the text is completely captured, the menu bar appears, where you can choose between different settings.



Touch the reading speed slider and move it to adjust the reading speed.



To quickly return to the original size within the magnification mode, press the zoom button.

If you do nothing for 10 seconds or briefly press the home button, the menu bar will disappear automatically.

Press and hold the home button to exit the TTS mode.



The image and / or the captured text can be saved by long pressing the freeze button. In picture mode, text and image will both be saved. The image is saved in the gallery under "Image", while the text is saved under "Document".



### Picture mode

In picture mode, the layout of the content remains the same as that of the captured picture. Zoom in/out, moving the image section and various color modes are still available.



If the image is not rotated properly, press and hold the camera switch button on the right-hand side to access the corresponding menu.



### Segmentation activated

When the segmentation function is activated, the content is automatically divided into several sections.



You can now choose between two types of reading. In the **Single selection** a section is highlighted and reading stops at the end of the section. In the **Multi-selection**, you can highlight several sections with your finger which are then read out one after the other.



Tap on the play or pause symbol in the menu bar or press the TTS button to start / stop reading the selected section.



Tap on the previous/next symbol to jump to the previous/next section and highlight it.



If playback is paused, you can select another section with a short tap.



A long press on a section will highlight it and reading will begin immediately.



## Segmentation deactivated

Here the recorded text is not divided into sections but recognized as continuous text and the active position in the text is highlighted. You can go to another part of the text with the forward/back buttons in the menu bar or via the touchscreen.



Tap on the play or pause symbol in the menu bar or press the TTS button to start / stop reading the selected sentence.



Tap on the previous/next symbol to jump to the previous/next sentence and highlight it.



Long press on a sentence to highlight it.



During playback, you can double-tap to jump to another point in the document and the reading will start from there.



## Text mode

In text mode you can only see the captured text and not the original image. You can still magnify the image, change the color mode and move the image section with your fingers or the joystick. Advanced functions such as TTS mode, freeze frame, ruler and mask are deactivated in this viewing mode.



To improve the contrast of the text, you can tap on the "A" in the menu bar and set the text display to bold.

If reading is on pause, you can see a scroll bar on the left side of the display. When you move the bar, the highlighted content changes accordingly.

Depending on which settings you have previously made in picture mode, you get different display and navigation options:

### Segmentation was activated in image mode



Tap the play or pause symbol in the menu bar or press the TTS button to start / stop reading the displayed section.



In the **Single-selection mode**, you will see the text of the active section and you can use the arrow keys to switch between all sections of the document. In the **Multi-selection mode**, you will see the text of the active section and you can use the arrow keys to switch between the sections selected in the picture mode.

### Segmentation was disabled in image mode

You see all of the text. The arrow keys are inactive and reading aloud starts at the beginning of the document.



Tap on the play or pause symbol in the menu bar or press the TTS button to start / stop reading.



Long press anywhere else in the document to start reading from this point.

## The main menu




If you press the home button for approx. 2 seconds, you will get to the main menu, where you can select the following functions:

- Magnifier
- Text to speech (TTS)
- Open document / picture
- settings

The selected application is highlighted. Select the icon on the touchscreen (or move the joystick to select and press the TTS button to confirm).

In the menu you can go back or exit the menu with the home button.

## Open and copy document / picture

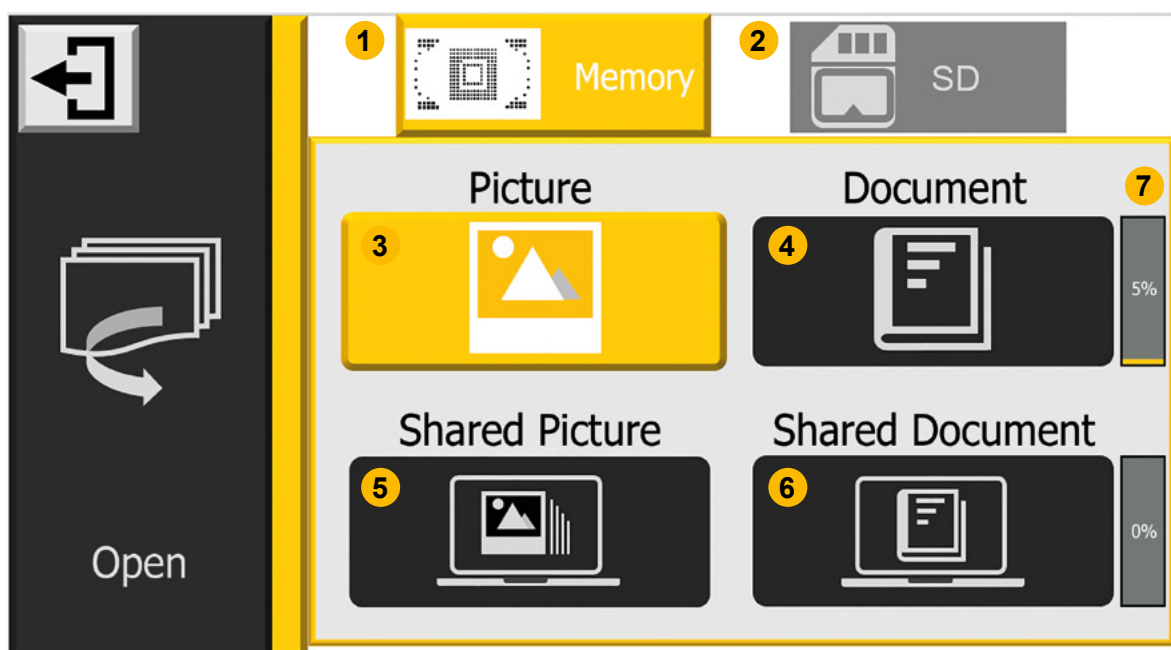

In this menu you can open pictures and text files, magnify them and have them read to you. You also have the option of copying saved files to an SD card. If you want to view external pictures, text files or PDFs on MANO EDU, you can copy them from a PC, laptop or an SD card to the device.

If you opened memory, you will see the folders of the internal memory(1) . Is an external SD card(2) inserted, you can view the content by pressing the symbol at the top right or turning the color mode button.

You can look at pictures and have the text captured for reading aloud, texts are displayed in text mode and can be viewed as described on page 24.

Briefly press the camera switch button to return to the magnifier.

## Manage files on the device

Here you can see four different folders in which the files are sorted by type and origin. In the first row you can open pictures(3) and documents(4) that you have previously saved by pressing and holding the freeze-frame button. In the second row you can view pictures(5) and texts(6) that you have previously copied from a PC or laptop to your device.

On the right side you can see the used memory in percent (7).

## Copy files from PC/laptop or SD card to the device



Connect the device to the PC via a USB-C cable (not included). MANO EDU is displayed as USB drive and you can copy files to the device.



In the open document/picture menu, open the SD card (2), select the desired files, and press the copy icon in the left column.

Supported picture formats are JPG, BMP, PNG, and TXT, PDF for documents. You will then find these files in the Shared Picture (5) and Shared Document (6) folders.

## Copy saved files to an SD card



Pictures and documents from the internal memory can be transferred to an external SD card if it is inserted. Select the desired picture / document in the internal memory and then tap on the SD card icon (or press the freeze frame button) to start the transfer.

Tap the SD card icon for 3 seconds (or press and hold the freeze button) to select all pictures / documents and transfer them to the external SD card.

When multiple selection is enabled, multiple pictures / documents can be transferred at once by tapping the transfer icon (or pressing the freeze button).

## Delete all files from the device



Go to the folders overview by selecting “Open” in the main menu. Press and hold the orientation aids button for approx. 5 seconds and confirm the deletion process by pressing it again or cancel the process by pressing any other button.

## Deleting files from a folder



Select the file you want to delete and tap the trashcan icon on the bottom left or press the orientation aids button. Press the button again to confirm the deletion. If you would like to cancel the deletion process, press another key or tap another part on the display.

If you want to delete all files in the current folder, press and hold the trashcan icon or the orientation aids button for approx. 5 seconds and confirm the deletion by pressing again or cancel the process as described above.

You can also delete certain pictures that you have previously marked in the multiple selection.

## Multiple selection

Long tap (or long press the TTS button) a specific picture or document in a folder to enable or disable multiple selections.

When multiple selection is enabled, a small box will appear in the lower right corner of each picture. Tap the box (or use the joystick and press the TTS button) to select all target pictures / documents. Then the multiple transmission or multiple deletion can be performed as described above.

Long tap the screen or tap the exit icon again (or press the home button or long press the freeze button) to exit multiple selections.

## Open pictures

Open the pictures folder on the device or SD card. There you can scroll up or down with the slider on the right side of the screen or with the joystick.

Tap the thumbnail to highlight it and tap it again (or press the TTS button) to view the picture. If the saved picture has a voice label, you can listen to it by tapping the voice symbol in the top right corner or by pressing the zoom button.

You can view and navigate withing the picture as described in section „Freeze frame function“ on page 19. Press the home button to open the menu bar



In the menu bar, tap on one of the arrows to go back / forward one picture.



In the menu bar, tap on one of the double arrows to go to the beginning or end of the pictures.



To rotate the picture, tap on the rotate icon in the menu bar. The last rotation is saved automatically.

Tap the exit icon (or long press the home button) to return to the gallery.

## Open documents

Here you will find the text and PDF files that you have previously saved in the reading mode (see page 23 ) or copied from a PC/SD card. If the text was saved together with the original picture in picture mode, the text is displayed in the gallery as a thumbnail in the corner of the saved picture. Press the magnification button to preview and adjust the magnification, color mode, etc. Press the home button to return to the gallery.

If the original picture is deleted from the pictures folder, a thumbnail can no longer be displayed in the gallery.

Double-tap the document to open it and view the captured text. Alternatively, you can open the document with the joystick and the TTS button.

Tap the exit icon (or long press the home button) to return to the gallery.

## View and read aloud text files



Tap the play or pause symbol in the menu bar or press the TTS button to start or stop reading.



By default, text capture is enabled in order to use the read aloud function. In this case, depending on the length of the text, it may take a few seconds for the text to be captured. If you do not need the read aloud function, deactivate text capture to turn the pages more quickly.



In pause mode, long press on a sentence to start reading from that point.



To improve the contrast of the text, you can tap on the "A" in the menu bar and set the text display to bold.



In longer text files, two icons automatically appear next to the playback button to scroll back/forward one page. Press and hold the key to turn back/forward first 10 and then 50 pages at a time. After turning the page, the text recognition for the new page starts. You can continue scrolling as soon as the text capture has been completed.

## View and read aloud PDF files



Tap one of the arrows in the menu bar to scroll back/forward one page. Press and hold to scroll back/forward first 10 and then 50 pages at a time.



Tap one of the double arrows in the menu bar to go to the beginning or end of the PDF..



To read aloud the current page, tap the read aloud icon in the menu bar and the page will be displayed in text mode.

## Settings

The last item in the main menu are the settings.

You can choose from the following submenus:



Options



language settings



Device information

Tap the submenu you want (or rotate the Color Mode button) to see the available options.

Tap on the item to change the settings (or press the joystick to highlight the item and press the TTS button to change it).

## Options page

### Individual color combinations

Here you can select which semi color combinations shall be displayed.

### Voice label

If the function is active, you can record a voice label when saving a freeze image.

### Power saving (standby mode)

If this function is active, the device is put into standby mode if it has not been used for 5 minutes. The device switches itself OFF completely if it has not been switched ON again after 15 minutes in standby mode.

### vibration

If this function is active, the screen vibrates briefly when you tap an icon on the touchscreen.

### Speech guide

Menus and functions are not only displayed but also described with voice prompts. You can choose between simple and full guide.

### Keystone Correction

We recommend that you activate this function, as it automatically compensates for distortions in the image if the display is at an angle to the reading material.

### Document scan

Here you can switch the function „Save multiple pages as a PDF“ (see page 19) ON and OFF.

## Language settings

### Male / female voice selection

If your language is set to English, you can also choose between US, UK and AU voices.

### Language selection

Here you can choose between the currently available languages. The setting changes the language in the menus and the voice output.

### Table mode

If you would like to capture the text of a table and have it read out, activate this function so that the order of the individual elements is displayed correctly.

## Device information

### Reset to factory settings

Here you can reset the device to the factory settings. After the reset, the device will automatically turn OFF.

### Frequency

Frequency offers the options 50 Hz and 60 Hz which can be changed depending on the region in which you are located. If the picture flickers, changing the frequency can lead to an improvement.

### Information about the system

Here you can find information such as the current firmware version.

## HDMI output

Connect the device to your external display device with an HDMI cable to enable dual display application.



## Hardware reset



If the device freezes and cannot be turned OFF the regular way, press the power button and both volume buttons at the same time to do a hardware reset.

## Handling of the battery

**Attention:** A prerequisite for a long battery life is the correct operation and charging of the battery.

Make sure the battery is fully charged before first use of the system. For charging, connect the power adapter with the power port on the left side of the system and plug it into an AC outlet.

If the magnifier is not used for a longer period of time, the battery loses its charge even if the device is switched OFF and a deep discharge and damage of the battery may occur. This can be avoided by fully charging the battery beforehand.



## Changing the battery

The battery can be removed from the device without tools. Turn the opened device upside down so that you can comfortably reach the bottom of the device. Press the release button on the back and the battery will be released. Take out the battery and insert a new one.

## **IV. Troubleshooting**

### **The device cannot be switched ON**

- Connect the device to the power adapter
- please charge the battery

### **The device cannot be switched OFF**

- Perform a hardware reset by pressing the power button and both volume buttons simultaneously.

### **Black screen or white screen**

- If the device is on a flat, clean surface, or if there is nothing on the base, the screen can be black or white.
- Adjust the magnification
- Set the color mode
- Restart the device

### **The picture doesn't change**

- The device may be in freeze image or playback mode
- Restart the device

### **Dirt or smudges on the screen**

- Adjust the brightness
- Clean the LCD screen and camera

### **The Picture is blurred**

- Check if the autofocus is deactivated
- Move your hand back and forth over the displayed document to refocus
- Tap the screen for 2 seconds to manually set the focus.

If the problem is not solved, reset the device to factory settings and perform a hardware reset. If you have any further problems, please contact Reinecker Vision GmbH

## **V. Warranty information**

1. Reinecker Vision GmbH grants a warranty of 24 months (warranty period only valid within the European Union) against defects in manufacture counted from date of invoice ex Reinecker Vision GmbH.
2. All parts that have been replaced when repairing the unit will automatically become property of Reinecker Vision GmbH.
3. Damage caused by improper use or by non-compliance with the operating instructions will result in the invalidation of the warranty.
4. Interventions from persons other than Reinecker Vision GmbH authorized specialists may lead to invalidity of warranty.
5. The liability for material defects does not include light sources serving to illuminate the reading object and other parts subject to wear and tear, e.g. rechargeable batteries.
6. If a warranty claim is necessary, the system has to be returned to the manufacturer in Alsbach-Hähnlein or to your authorized dealer in the original packaging including all components. The manufacturer does not carry the cost for transport, packaging and insurance. Prior to returning the system, a RMA number issued by Reinecker Vision is required. Please have the exact serial number of your system before hand.

Phone number: +49 6257 9311-0

[View our General Terms and Conditions.](#)

## VI. Technical details

|                              |                                                                                                        |
|------------------------------|--------------------------------------------------------------------------------------------------------|
| <b>Type</b>                  | MANO EDU (MANOEDU1/ MANOEDU1LV)                                                                        |
| <b>Display</b>               | FullHD 12.5" (approx. 31.8 cm) touch display on a tiltable stand, screen resolution: 1920 x 1080       |
| <b>Cameras</b>               | Ultra HD 4K camera, video resolution: 3840 x 2160                                                      |
| <b>Zoom</b>                  | Document view: 1x - 60x<br>(depending on the viewing angle)                                            |
| <b>Color mode</b>            | True colors, up to 16 semi colors adjustable                                                           |
| <b>Freeze image function</b> | Image storage, storage capacity: approx. 2 GB                                                          |
| <b>External file formats</b> | Image formats: JPG, BMP, PNG<br>Document formats: TXT, PDF                                             |
| <b>Rechargeable battery</b>  | exchangeable Li-Ion battery                                                                            |
| <b>Operating time</b>        | up to approx. 4.5-5.5 hours<br>(depending on the application)                                          |
| <b>Charging time</b>         | approx. 3 hours                                                                                        |
| <b>Power adapter</b>         | Input: 110-240V; Output: 12V / 3A                                                                      |
| <b>Dimensions</b>            | set up approx. 38 x 25 x 26 cm/15 x 9.8 x 10.2 in,<br>folded approx. 38 x 4 x 26 cm/15 x 1.6 x 10.2 in |
| <b>Weight</b>                | 2.4 kg/5.3 lbs (including battery)                                                                     |







**Manufacturer**

**Reinecker Vision GmbH  
International Contact**

**sales department, customer relations:**

Sandwiesenstraße 19  
DE-64665 Alsbach-Hähnlein

**customer service:**

Erlenweg 3  
DE-64665 Alsbach-Hähnlein

Tel.: +49 6257 9311-0

Fax: +49 6257 9311-8230

Email: [international@reineckervision.de](mailto:international@reineckervision.de)  
[www.reineckervision.com](http://www.reineckervision.com)

SRN: DE-MF-000025998

In case service is necessary, please  
contact your authorized distributor.



**Hersteller**

**Reinecker Vision GmbH  
Zentrale**

**Vertrieb, Kundenberatung:**

Sandwiesenstraße 19  
64665 Alsbach-Hähnlein

**Kundendienst:**

Erlenweg 3  
64665 Alsbach-Hähnlein

Tel.: 06257 93 11-0

Fax: 06257 93 11-8008

Email: [info@reineckervision.de](mailto:info@reineckervision.de)  
[www.reineckervision.de](http://www.reineckervision.de)

SRN: DE-MF-000025998

Bitte wenden Sie sich bei Fragen  
oder einem Servicefall direkt an unser  
Hauptwerk in Alsbach-Hähnlein.

Bei im Ausland erworbenen Produkten  
kontaktieren Sie im Servicefall bitte  
Ihren dort ansässigen Händler.