


# quick reference guide – MANO EDU

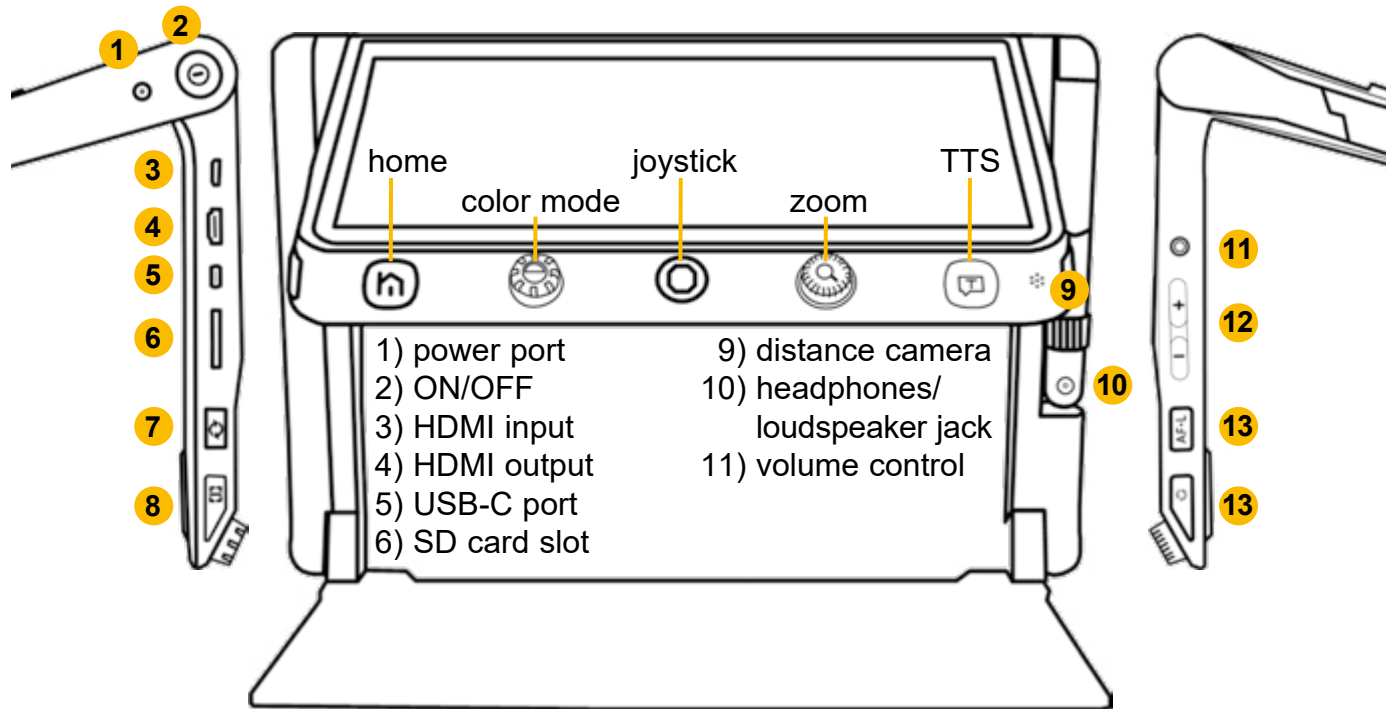

- |                |                     |                                  |
|----------------|---------------------|----------------------------------|
| 1) power port  | 6) SD card slot     | 11) headphones/ loudspeaker jack |
| 2) ON/OFF      | 7) camera switch    | 12) volume control               |
| 3) HDMI input  | 8) line marker/mask | 13) autofocus                    |
| 4) HDMI output | 9) microphone       | 14) freeze button                |
| 5) USB-C port  | 10) distance camera |                                  |

**A**



- A** Hold the base plate with one hand and unfold the stand

- B** Bring the display into a comfortable viewing position

- C** Unfold the distance camera and turn it in the desired direction

**B**



**C**



# quick reference guide – MANO EDU

| Operation in magnification mode and menus                                           |                                                                                                                                                     |  |                                                                                                                                                              |
|-------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------|--|--------------------------------------------------------------------------------------------------------------------------------------------------------------|
|    | short press: battery status                                                                                                                         |  | long press: switch ON/switch OFF/standby                                                                                                                     |
|    | short press: show/hide menu bar                                                                                                                     |  | long press: open menu                                                                                                                                        |
|    | turn controller: change color mode                                                                                                                  |  | press controller: switch between true colors and artificial colors                                                                                           |
|    | horizontal and vertical navigation in magnification mode; navigation within the picture gallery; adjust size and position of ruler and line masking |  |                                                                                                                                                              |
|    | turn controller: zoom in/out                                                                                                                        |  | press controller: overview mode                                                                                                                              |
|    | zoom in/out: use the pinch and zoom gesture                                                                                                         |  |                                                                                                                                                              |
|    | tap on the magnified image with your fingers and move it                                                                                            |  |                                                                                                                                                              |
|   | short press: switch the camera image<br>long press: enter A4 reading mode                                                                           |  |                                                                                                                                                              |
|  | short press: switch between line marker and mask                                                                                                    |  |                                                                                                                                                              |
|  | short press: freeze image                                                                                                                           |  | long press: save image                                                                                                                                       |
|  | short press:<br>start reading (current image section)<br>short press in menus and galleries:<br>confirm selection                                   |  | long press: start reading (A4 page)<br><br>long press in galleries: multiple selection                                                                       |
|  | in menus and galleries: tap with your finger on an icon to select it                                                                                |  |                                                                                                                                                              |
| Operation in TTS mode                                                               |                                                                                                                                                     |  |                                                                                                                                                              |
|  | short press:<br>show/hide menu bar<br>long press: exit TTS mode                                                                                     |  | <br>turn controller:<br>zoom in/out<br>press controller:<br>original zoom |
|  | switch between image-<br>and text mode                                                                                                              |  | <br>text segmentation ON/OFF                                              |
|  | start/stop reading                                                                                                                                  |  | <br>back/forward<br>by segment/by sentence                                |
|  | adjust reading speed                                                                                                                                |  | <br>in image mode: save image<br>in text mode: save text                  |